


AVOXimeter® Whole Blood Oximeter 1000E

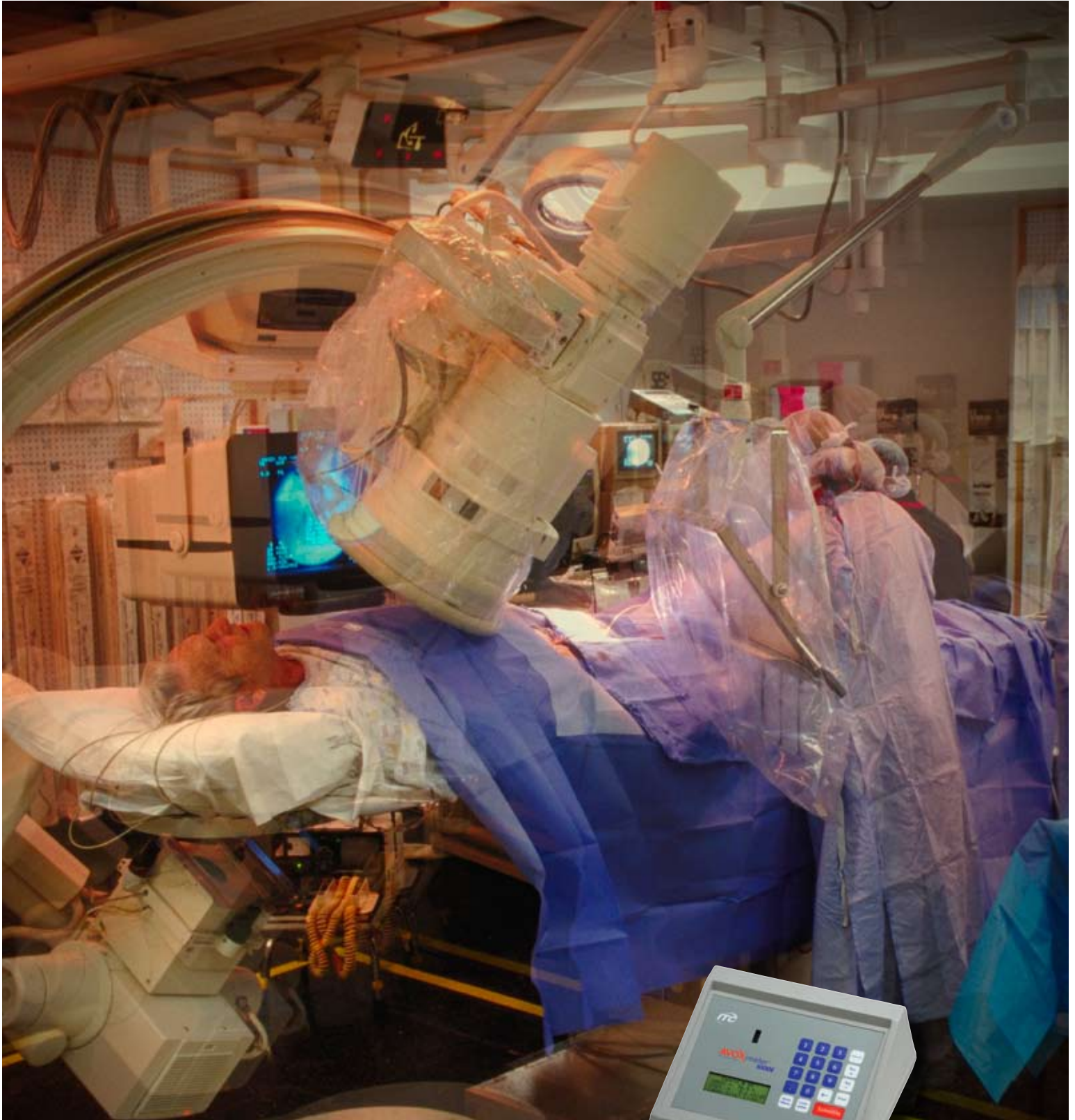

THE POINT OF CARE

Rapid, Accurate Oximetry assessment

Today's clinicians seek technology that helps to provide a more accurate and timelier diagnosis during cardiac catheterization procedures. The portable AVOXimeter® 1000E is an advanced oximeter that efficiently measures hemodynamic function for a more comprehensive picture. The AVOXimeter 1000E measures total hemoglobin concentration, oxyhemoglobin fraction and computes oxygen content in less than 10 seconds.

RAPID, RELIABLE OXIMETRY ASSESSMENT

- Patented state-of-the-art optics use multiple wavelengths to yield accurate measurements of:
 - Fractional O₂ Saturation
 - O₂ Content
 - Total Hemoglobin Concentration
- Quantitative results in less than 10 seconds
- Single-use cuvettes for maintenance-free convenience and optimal instrument performance



```
Sample # 1
%HbO2 = 98.6 %
[O2] = 16.4 ml/dl
[THb] = 11.9 g/dl_
```

```
**Cardiac Output**
O2 Uptake=250_ml/min
Venous[O2]=14.0ml/dl
Art.[O2]=19.0ml/dl
```

Easy to Read Display

SAFE, CONVENIENT AND EASY TO USE

- Simple, two step test method:
 - inject sample into cuvette, and
 - insert cuvette into the instrument
- No sample preparation required, uses whole blood
- Room temperature cuvette storage with no expiration date



- for the Cardiac Cath Lab



IDEAL FOR ADULT AND PEDIATRIC CARDIAC CATH CASES

- Quantitative measurements aid in diagnosing and detecting intracardiac and great-vessel shunts
- Ideally suited to trend “Step-Ups” in O_2 Saturation while advancing catheters in right heart procedures
- Supports blood conservation protocols, uses only 50uL of whole blood
- Stores up to 500 samples for post procedural case documentation

OPTIMIZED FOR THE CARDIAC CATHERIZATION LAB

- Measures and displays differences in oxygen saturation between adjacent anatomical sites
- Utilizes the Fick Principle to calculate cardiac output
- Computes and displays the following:

Measured Oxygen and Hemodynamic Calculations

- | | |
|--|---|
| • Fractional Oxygenation Saturation | • Stroke Volume |
| • O_2 Content | • Stroke Index |
| • Total Hemoglobin Concentration | • Pulmonary Blood Flow |
| • Body Surface Area | • Total Systemic Resistance |
| • Estimated Oxygen Uptake | • Total Pulmonary Resistance |
| • Cardiac Output (Systemic Blood Flow) | • Pulmonary-to-Systemic Flow Ratio (from flows) |
| • Cardiac Index | • Pulmonary-to-Systemic Flow Ratio (from oxygen saturation) |

REGULATORY COMPLIANCE AND DATA MANAGEMENT DRIVEN

- Results stored in memory include patient and operator identification number, date, time, LQC lot number and level, anatomical site sampled, and test result
- User defined security features including QC, PID and OID lockouts
- Quick and easy two-point optical quality control
- Data storage and ability to print report or transfer to data management system, OxyReview™ and/or the LIS



Data Management/Connectivity

MEASUREMENTS AND OPERATING RANGES

Measurement	Operating Range	Accuracy	Precision	CPT Code
Fractional O ₂ Saturation	0 - 100%	1%	0.5%	82180
Total Hemoglobin Concentration	4 to 25 g/dL	< 10g/dL = 0.35 g/dL > 10g/dL = 0.45 g/dL	0.3 g/dL	85018
Oxygen Content (O ₂ Ct)	0 - 35 mL O ₂ /dL			

NO CLINICALLY SIGNIFICANT INTERFERENCES FROM:

- Bilirubin
- Hemolysis
- Carboxyhemoglobin
- Methemoglobin
- Fetal Hemoglobin

AVOXimeter 1000E ORDERING INFORMATION

Product	Product Code
AVOXimeter 1000E 110V	AVOX1000E-110
AVOXimeter 1000E 220V	AVOX1000E-220
AVOXimeter Cuvettes (100 cuvettes)	C100B
Dymo Label Writer SE 300 Printer 110V	SE300-110
Dymo Label Writer SE 300 Printer 220V	SE300-220

Optional Printer



To learn more about how the AVOXimeter 1000E can help you improve patient care at the bedside, please contact us at 1.800.631.5945 or visit www.itcmed.com

2007 ITC. All rights reserved. ITC and AVOXimeter are registered trademarks of International Technidyne Corporation in the United States and other jurisdictions.



8 Olsen Avenue • Edison, NJ 08820 USA
tel: 732.548.5700 • fax: 732.248.1928
www.itcmed.com

a subsidiary of Thoratec Corporation